EasySep™ Buffer

Cell separation buffer

REF #100-0780 1 L

Document #1000016106 | Version 01

Scientists Helping Scientists™ | WWW.STEMCELL.COM

TOLL FREE PHONE 1 800 667 0322 • PHONE +1 604 877 0713
INFO@STEMCELL.COM • TECHSUPPORT@STEMCELL.COM
FOR GLOBAL CONTACT DETAILS VISIT OUR WEBSITE

ENGLISH

Intended Use

EasySep™ Buffer is intended to be used with EasySep™ IVD cell isolation kits for washing of cells or as diluent during enrichment of targeted cells for downstream in vitro diagnostic assays.

Product Description

EasySep™ Buffer is recommended for use in most manual EasySep™ cell separation protocols. It contains Dulbecco's phosphate-buffered saline (PBS) with 2% fetal bovine serum (FBS) and 1 mM EDTA. It has a suitable, physiologically relevant pH optimized to maintain cell survival. It also has osmolality similar to the natural environment of the cells to maintain balanced osmotic pressure.

Properties

Storage: Store at 2 - 8°C.

Shelf Life: EasySep™ Buffer is stable until the expiry date (use-by date) on the label.

Warnings and Precautions

1. FOR IN VITRO DIAGNOSTIC USE.
2. Refer to the Product Information Sheet (PIS) of the EasySep™ cell isolation kit for the list of materials required and detailed instructions on the use of this product with the kit for the enrichment of targeted cells.
3. This product should be handled by trained personnel observing good laboratory practices. Once this product is added to human cells, treat the suspension as potentially biohazardous. Handling and disposal of wastes should observe all local, state, or national regulations.
4. Ensure that bottle is capped properly during storage. Proper handling and storage of this product is essential. Shelf life and performance may be adversely affected when the bottle is stored outside the recommended storage conditions.
5. Do not use the product if packaging container's integrity has been compromised upon arrival or if there are any signs of contamination, cloudy appearance, discoloration, drying, leaks, or any other evidence of deterioration. Unused product may be disposed of according to standard laboratory procedures for non-hazardous liquids.
6. Do not use the product beyond the use-by date indicated on its label.
7. Do not freeze.
8. Important information regarding the safe handling, transport, and disposal of this product is contained in the Safety Data Sheet (SDS).
NOTE: SDS of this product is available at www.stemcell.com.

Directions for Use

This product has been optimized for use with EasySep™ cell separation kits. Refer to the PIS for the applicable cell separation kit or contact STEMCELL for more information.

STEMCELL Technologies Canada Inc. | 1618 Station Street, Vancouver, BC | V6A 1B6 | Canada | www.stemcell.com

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Document #1000016106
Version 01
Issue Date 2024-02

Technical Assistance

This PIS is also available in various national languages in electronic format on STEMCELL's website and can be provided in paper copy upon request.

For technical support, contact techsupport@stemcell.com or call toll-free either +1.604.877.0713 (Canada), +1.800.667.0322 (North America), or 00800 7836 2355 (Europe). For more information, visit www.stemcell.com.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the member state in which the user is established.

Catalog or reference number	Batch code	Use-by date: YYYY-MM-DD
Caution, consult accompanying documents	In Vitro Diagnostic Medical Device	For storage within temperature limits
Consult Instruction for Use or electronic Instructions for Use	Do not use if packaging is damaged	Unique device identifier
CE Mark	Manufacturer	Authorized EC representative in the European Community

EASYSEP™ BUFFER IS MANUFACTURED IN COMPLIANCE WITH 21 CFR PART 820 AND ISO 13485. FOR ADDITIONAL INFORMATION ON QUALITY AT STEMCELL, REFER TO WWW.STEMCELL.COM/COMPLIANCE.

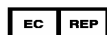
Copyright © 2024 by STEMCELL Technologies Inc. All rights reserved including graphics and images. STEMCELL Technologies & Design, STEMCELL Shield Design, Scientists Helping Scientists, and EasySep are trademarks of STEMCELL Technologies Canada Inc. All other trademarks are the property of their respective holders. While STEMCELL has made all reasonable efforts to ensure that the information provided by STEMCELL and its suppliers is correct, it makes no warranties or representations as to the accuracy or completeness of such information.

STEMCELL Technologies Canada Inc. | 1618 Station Street, Vancouver, BC | V6A 1B6 | Canada | www.stemcell.com

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MDSS GmbH